

ALCOLIMIT Instructions

(Read full instructions before use)

1. Turning on the power

Slide off the battery cover and remove the battery protector strip.

To turn the unit on, depress the power button for 2 seconds. The unit will commence a start up process and automatically re-stabilize the alcohol sensor. The LCD display will undergo a count down sequence, indicating that purge cycles and circuit tests are taking place. This process generally takes 30 seconds but can last 2 minutes. The unit is ready when two beeps sound and the "WAIT" signal disappears off the screen.

0,00 will be displayed on the LCD to indicate the start up process has been completed.

2. Blow in the unit

Ensuring your mouth is close to the intake sensor (about 2 cm), blow one deep breath for at least 4 seconds.

Please note: Alcohol vapour on your breath will cause a higher reading. We recommend having a glass of water prior to testing and waiting at least 20 minutes after your last drink for more accurate results.

3. Display

Range	Digital Display and Audio Signals
0,00% - 0,01%	1 beep
0,02% - 0,04	2 beeps
0,05% and above	5 second beep


ALCOLIMIT has a range of 0,00-0,40 %BAC

4. Retesting

When "WAIT" sign on the LCD is no longer illuminated and 0,00 is displayed on LCD, the unit is reset and ready for retesting.

After 5 minutes of non-use the unit automatically turns off.

5. Low Battery indicator

ALCOLIMIT lets you know when the battery is low. The  signal is displayed on the LCD during use. Fresh batteries (2 x "AA" size) should be installed before retesting.

Note: the low battery signal flashes briefly during start-up. This does not mean the battery is low.

WARNINGS

Closed rooms with poor ventilation, pollution or air conditioning may delay the time for resetting between tests.

To obtain accurate results, wait at least 20 minutes after your last drink.

Have a glass of water or other non-alcoholic beverage to minimise alcohol vapour in your mouth as the vapour can cause a higher reading.

Retest 2-3 times to obtain reliable test result.

Do not blow cigarette smoke or liquid into the tester.

Do not use ALCOLIMIT as a tool to drink and drive. There is no safe level of alcohol consumption before driving or operating machinery.

Keep out of extreme temperatures and reach of children.

To maintain accuracy recalibrate unit every 6-12 months depending on use. For details on our recalibration see our website: www.alcolimit.com.au

WARRANTY

Manufacturer warrants for 90 days the product to be free from defects in workmanship or materials (excluding calibration) in normal service from the date of purchase.

Manufacturer's obligation under this warranty is limited to replacing, adjusting, or repairing the unit if returned along with proof of purchase.

This warranty is void if unit has been tampered with, maliciously damaged or used for any purpose other than that intended.

DISCLAIMER

The manufacturer, importer or distributor takes no responsibility whatsoever for the use of this product for any reason. This product must not be used as a tool for determining whether a person is able to operate a vehicle or machinery. The intake of any alcohol will impair reflexes and judgment to operate motor vehicles and machinery.